


THE REVOLUTION CHAIR

Smart

Seating

Aged and Health Care Seating Solutions

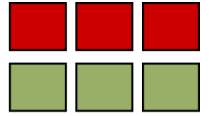

1. Position chair facing table so arms are 35mm from table edge. Make sure canted legs are positioned to the front. This provides stability when seat is extended forward.
2. Turn seat left or right to meet user. Seat will lock for safety. When seated, lift swivel lever (A) and rotate chair to face table.
3. Lift slide lever (B) and move seat forward. Release lever and seat will lock in first position. To move closer to table, lift lever, slide forward to second position, then release lever again.
4. To exit chair, lift slide lever and glide seat back. Seat rotates left or right, locking for safe exit.